


# ConnectClip

## Quick Start Guide

### PAIRING CONNECTCLIP WITH HEARING AIDS

1. **Ensure ConnectClip is powered off.** Press and hold multi-button for approximately 6 seconds until power indicator OFF ●
2. **Turn both hearing aids OFF.** (Open battery drawer)
3. **Turn both hearing aids ON.** (Close battery drawer)
4. **The hearing aids will now be ready** for first time connection with the ConnectClip. You now have 3 minutes to complete the pairing.
5. **Turn ON ConnectClip.** Press and hold multi-button for approximately 3 seconds until power indicator shows STEADY GREEN ●
6. **ConnectClip will now search for your hearing aids.** Status indicator BLINKING YELLOW ●
7. **Place ConnectClip and both hearing aids close to each other** (maximum distance is 8 inches)
8. **Connection established with hearing aid(s)** is indicated by status indicator STEADY YELLOW ●

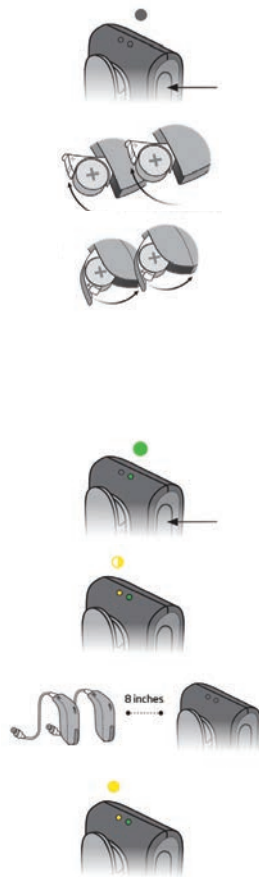

### PAIRING CONNECTCLIP WITH MOBILE PHONE

1. **Pair the ConnectClip with your hearing aids before pairing ConnectClip with your mobile phone.**
2. **Place ConnectClip in Pairing mode.** Press and hold the multi-button AND volume up button for about 6 seconds (A) until status BLINKS BLUE ●
3. **Enable pairing on mobile phone.** Turn on Bluetooth. Search for and connect to/pair with ConnectClip. This is normally in the Settings/Bluetooth section of your phone.
4. **Enter your pin code.** If your phone requests a pin code enter 0000 (four zeros).
5. **Pairing complete.** Pairing complete with mobile phone is indicated by status STEADY BLUE ●



### RECEIVING CALLS

**Phone connected:** When the ConnectClip is turned on and is in range of your mobile phone it will automatically connect to the mobile phone. The status indicator is STEADY BLUE ●


**Phone ringing:** When the phone rings you will hear a ring indicator (and, if available, the ring-tone from your phone) in your hearing aids. The status indicator will BLINK GREEN ●



**Answer call:** Either a short press of the multi-button (A) – or answer the call from your mobile phone.

Once the call is connected status will change to STEADY GREEN ●

**End call:** Either a short press of the multi-button (A) – or hang up the call from your mobile phone. Status STEADY BLUE ●

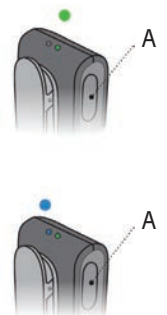

### LISTENING TO MUSIC

If connected to a mobile phone or other paired device through Bluetooth, the music played on the phone/device will automatically be played in your hearing aids. STATUS ORANGE ●

**You do not need to press any buttons or change anything on the ConnectClip.**




## REMOTE MICROPHONE

Press and hold multi-button for approximately 3 seconds to change mode. Mode change is confirmed by an audible indication in your hearing aids. STATUS MAGENTA ●

See "ConnectClip Quick Guide to the Remote Microphone" for more details.

**IMPORTANT:** Ensure that the microphone is pointing to the speaker's mouth and that it is not covered by clothing, scarf, tie, etc.

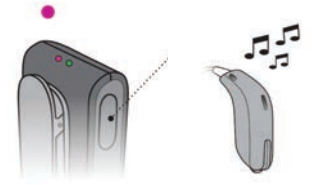

## REMOTE CONTROL

When ConnectClip is not streaming sound it doubles as a remote control for your hearing aids.



- **Volume up**  
– short press
- **Volume down**  
– short press



- **Next program**  
– Press and hold for 3 seconds for next program
  - **Mute hearing aid microphones**  
– long press on volume down button
- Short press volume up or down will **unmute**

**IMPORTANT:**  
When used as a remote control, the maximum distance between ConnectClip and hearing aids is 6 feet.

## Overview – ConnectClip blinking patterns

Status	Description
	Blinking yellow Searching for hearing aids
	Steady yellow Hearing aids connected. Not connected to a phone.
	Blinking blue Searching for mobile phone (pairing)
	Steady blue Connected to a mobile phone
	Steady green On a call
	Blinking green Incoming call
	Steady orange Music, Mobile Phone
	Steady magenta Remote microphone mode
	Blinking magenta Remote microphone reconnecting

Power	Description
	Steady red Battery low
	Steady green Turned on – not charging
	Blinking green Turned on – charging
	Fading green Turned on – charging complete
	Blinking orange Turned off – charging
	Steady orange Turned off – charging complete

14242 15555-10192 / 12.17